

AMS INTERNATIONAL

**The Old Vicarage
Tuxford
Newark
Notts
NG22 0HP**

Toothbrushing Programmes

S-RACKS & LIDS

Cleaning Instructions

02/09/2020

- **Rinse with hot water**
- **Clean in warm/hot soapy water using standard detergents (Maximum temperature = 75°C) or cold sterilize**
- **Dishwash on top shelf only**
- **Do not steam, boil or microwave**
- **Air-dry or dry with clean paper towels**

**Before first use wash thoroughly.
Clean and dry weekly (more if soiled)**

S-Racks and lids should be replaced if cracks, scratches or rough surfaces develop.

**Tel: 01777 872015 Fax: 01777 870057
E-mail: info@amsinternational.net
www.amsinternational.net
VAT No. 684 1115 44**